


I.V. House 939L-Ultra, 939L-W-Ultra or 939XL-W-Ultra TLC® Wrist Splint with Straps / Directions

Single Use Only—Fits either wrist.


The TLC Wrist Splint can be applied to the wrist before or after inserting IV and connecting to extension set or IV tubing.

1.

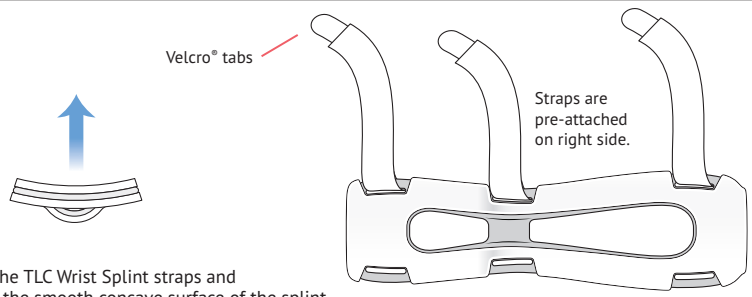


Unroll the TLC Wrist Splint straps and identify the smooth concave surface of the splint. This concave surface will be placed against the patient's hand and forearm.

2.

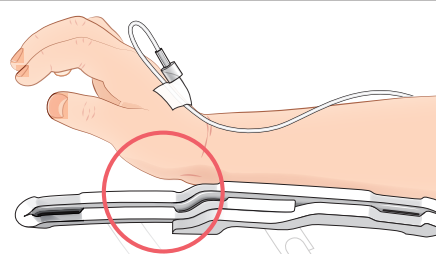


Place the splint under the patient's forearm, with the heel of the hand landing in the slight bend in the device. This places the hand in a functional position.

3.

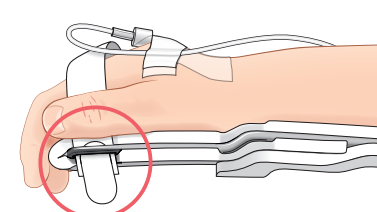


Wrap the first strap around the fingers. Thread the Velcro® tab into slot and pull it through. Fold the strap around the outer edge of the splint and back onto itself. Secure with Velcro tab.

4.

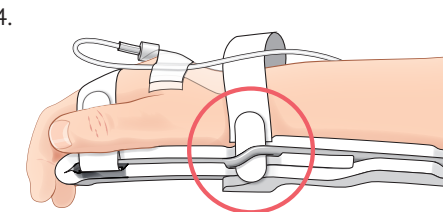


Repeat at the wrist. Wrap the strap around wrist. Thread the Velcro tab into slot and pull it through. Fold the strap around the outer edge of the splint and back onto itself. Secure with Velcro tab.

5.

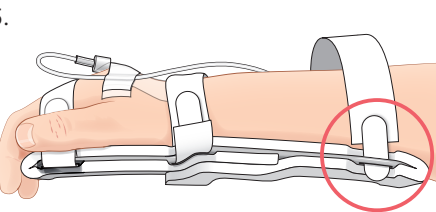


Repeat at forearm. Wrap the strap around forearm. Thread the Velcro tab into slot and pull it through. Fold the strap around the outer edge of the splint and back onto itself. Secure with Velcro tab.

6.

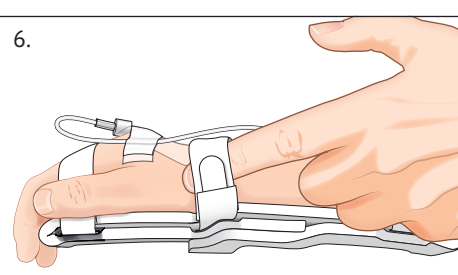

Note: Slip one finger beneath each strap to ensure the fabric is not so tight that it impedes circulation or infusion of IV fluids.

7. Optional Strap Configuration

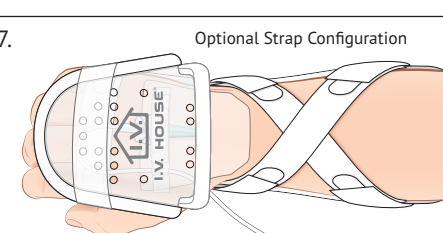

If the wrist strap obscures the catheter insertion site, consider a crisscross strap configuration instead. Put the wrist strap through the forearm slot and the forearm strap through the wrist slot. This will allow easy visual inspection of the catheter insertion site.

Notes:

- Protect the catheter and loop of tubing** with an I.V. House UltraDressing® or I.V. House UltraDome®.
- Check the IV site hourly** for redness, swelling, and tenderness.
- Touch, Look, and Compare** the IV site with the opposite palm and forearm. Change the TLC Wrist Splint each time a new IV is inserted.



939L-Ultra TLC® Wrist Splint
Fits most children ages 3–10 years,
or weighing 12–41 kg (26–90 lb.)



939L-W-Ultra TLC® Wrist Splint
Fits most children ages 3–10 years,
or weighing 12–41 kg (26–90 lb.)



939XL-W-Ultra TLC® Wrist Splint
Fits most youth/adults ages 10 years and up,
or weighing over 40 kg (88 lb.)

Touch / Look / Compare



I.V. HOUSE®
Protection Over and Above®

I.V. House, Inc. / St. Louis, MO 63146 USA

314-453-9200 / 800-530-0400 / FAX 314-453-9576 / I.Vinfo@ivhouse.com / www.ivhouse.com