


# I.V. House TLC® Elbow Splint with Straps

## IV Therapy Elbow Splint / Directions

Single Use Only – Fits either arm.


The TLC Elbow Splint can be applied to the arm before or after inserting IV and connecting to extension set or IV tubing.

**1.**

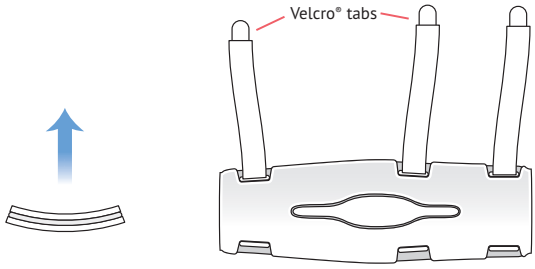


Unroll the TLC Splint straps and identify the smooth concave surface of the splint. This concave surface will be placed against the patient's arm.

**2.**

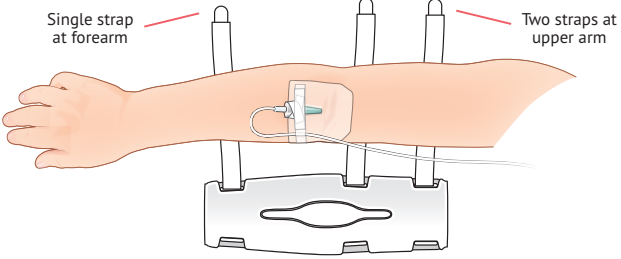


Place the splint under the patient's arm, with the narrow end of the splint toward the hand and the wide end above the elbow. The point of the elbow should align with the center opening.

**3.**

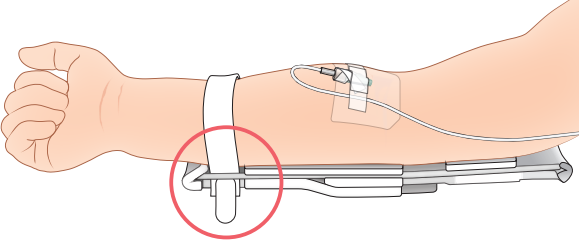

Wrap the first strap around the forearm. Thread the Velcro® tab through the slot on the opposite side of splint. Pull the tab through the slot, fold the strap back around the outer edge of the splint onto itself, and secure with the Velcro tab.


**4.**

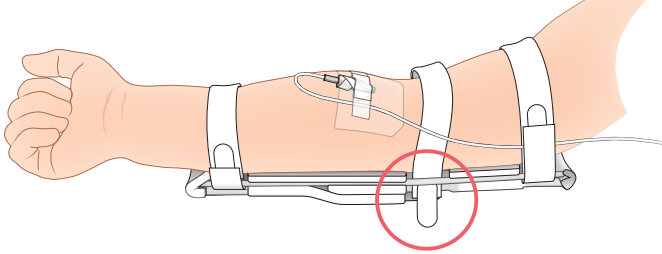

Repeat with the other two straps, securing each with the Velcro tab.  
**Note:** Slip one finger beneath each strap to ensure the fabric is not so tight that it impedes circulation or infusion of IV fluids.

**5.**

Optional Strap Configuration

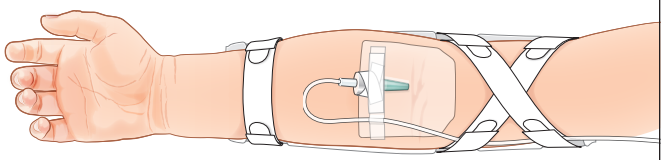

If the elbow strap obscures the catheter insertion site, consider a crisscross strap configuration instead. Put the elbow strap through the upper arm slot and the upper arm strap through the elbow slot. This will allow easy visual inspection of the catheter insertion site.

**Notes:**

- Protect the catheter and loop of tubing** with an I.V. House UltraDressing® or I.V. House UltraDome®.
- Check the IV site hourly** for redness, swelling, and tenderness.
- Touch, Look, and Compare** the IV site with the opposite palm and forearm. Change the TLC Wrist Splint each time a new IV is inserted.

**XS** 959XS-Ultra TLC® Elbow Splint  
Fits most newborns ages 0–4 months,  
or weighing 2.5–7 kg  
Small splint, shorter straps

**S** 959S-Ultra TLC® Elbow Splint  
Fits most infants ages 4–12 months,  
or weighing 6–12 kg  
Small splint, longer straps

**M** 959M-W-Ultra TLC® Elbow Splint  
Fits most toddlers ages 12–36 months,  
or weighing 9–15 kg  
Large splint, shorter straps

**L** 959L-W-Ultra TLC® Elbow Splint  
Fits most children ages 3–10 years,  
or weighing 12–41 kg  
Large splint, longer straps

Touch / Look / Compare