



**I.V. HOUSE®**

Protection Over and Above®

**M**

# 330M I.V. House UltraDressing®

**1 Smaller Dome Size IV Site Protector - 330 Series**  
**1 Sheet of 4 Puppy Stickers**

U.S. Patents Nos.  
6,526,981 / 8,006,699 / 8,640,707  
Canadian Patent No. 2,415,102  
Additional Patents Pending  
All Rights Reserved • Made in USA

Video Directions and Info:  
[www.ivhouse.com/how](http://www.ivhouse.com/how)

Manufactured for I.V. House, Inc.  
2072 Concourse Dr.  
St. Louis, MO 63146 USA  
[IVinfo@ivhouse.com](mailto:IVinfo@ivhouse.com) (E-mail)

800-530-0400 (toll free)  
314-453-9200 (phone)  
314-453-9576 (fax)



Wellkang Ltd, Enterprise Hub,  
NW Business Complex, 1 Beraghmore Rd,  
Derry, BT48 8SE, Northern Ireland



SINGLE USE



MRI SAFE



LATEX FREE



DEHP  
DEHP FREE



BPA FREE

CATALOG ITEM: **REF** 330M

**LOT** 2024-01-01-330M REV 0124



(01)0068600400071

# 330M I.V. House UltraDressing®

## Directions for Use— Single Use Only

1. **Protect the loop of 3" extension tubing** and catheter hub with the plastic dome.
2. **Wrap comfortably**, leaving enough stretch to fit a finger under the fabric.
3. **Cut off excess fabric.**
4. **Rewrap the fabric.** Close with Velcro®.
5. **Inspect the IV site** by releasing the Velcro tab, lifting the dome, and viewing the entire site.
6. **Change the I.V. House UltraDressing** for each new IV insertion or when soiled.

## Puppy Sticker

Remove the crack and peel backing from the puppy sticker and apply the sticker to the dome.

Tell the patient that the puppy likes to sleep on the “house” and to leave her alone.

## Note:

**It is important that the looped tubing is covered by the dome** to prevent accidental snagging of the loop and catheter hub. This also prevents bending and occlusion. Covering both the hub and the looped tubing virtually eliminates catheter manipulation.



For Detailed Directions  
[www.ivhouse.com/how](http://www.ivhouse.com/how)