



I.V. HOUSE®

Protection Over and Above®

750LFP

750LFP I.V. House UltraDome® Plus IV Site Protector

1 Large Foam Padded Dome IV Site Protector Plus Open Edge Padding
1 Puppy Sticker

Directions for use on back of insert

Manufactured for I.V. House, Inc., 2072 Concourse Dr., St. Louis, MO 63146 USA

IVinfo@ivhouse.com (E-mail)

www.ivhouse.com

800-530-0400 (Toll Free)

314-453-9200 (Phone)

314-453-9576 (Fax)

All Rights Reserved

Made in USA



(01)00686004000323



Wellkang Ltd, Enterprise Hub,
NW Business Complex, 1 Beraghmore Rd,
Derry, BT48 8SE, Northern Ireland



CATALOG ITEM: **REF** 750LFP

LOT 2024-04-01-750LFP REV 0424



SINGLE USE



MRI SAFE



LATEX FREE



DEHP
DEHP FREE



BPA FREE

750LFP I.V. House UltraDome® Directions

Single Use Only

1. Following IV insertion, **tape the loop of tubing** small enough to fit entirely inside the inner curve of the dome.
2. **Place an I.V. House UltraDome** over the catheter hub and transparent dressing.
3. **Tape across the I.V. House UltraDome** to hold the protective dome in place. Allow the tubing to exit the open end.
4. Lift the tape from one side to **inspect the IV site** according to your nursing practice guidelines.
5. **Change the I.V. House UltraDome** each time a new IV is inserted.

Puppy Sticker

Remove the crack and peel backing from the puppy sticker and apply the sticker to the dome.

Tell the patient that the puppy likes to sleep on the “house” and to leave her alone.



For Detailed Directions
www.ivhouse.com/how