

**FOOT / NEWBORN / 2.5-7 KG**

# TLC<sup>®</sup> Foot Splint

Touch/Look/Compare



## 949XS-Foot

1 Extra-Small (Newborn) Size Foam Padded Foot Splint with Two Straps and Velcro<sup>®</sup> Closure

Fits most infants ages 0-4 months, or weighing 2.5-7 kg

CATALOG ITEM:

**REF** 949XS-Foot



(01)00686004000255

**LOT** 2024-04-01-949XS-Foot REV 0424



**I.V. HOUSE<sup>®</sup>**

Protection Over and Above<sup>®</sup>

# 949XS TLC® Foot Splint—Directions

**Single Use Only—Fits either foot, laterally or medially**

- 1. Orient the concave side of the splint facing upward** and unroll the straps.
- 2. Align the infant's ankle with the splint's middle opening.** The heel should roughly coincide with the prominent curve at the outer edge of the splint.
- 3. Rotate the splint to position it** laterally or medially, on a left or right foot.
- 4. Leg Strap:** Slide strap under IV tubing and wrap the top strap around the lower leg and the splint. Secure tubing at lower leg with final pass of strap. Fasten with Velcro® tab.
- 5. Toe Strap:** Wrap the bottom strap over the toes, under the loop of tubing, and slip the free end through the hole in the splint near the toes. Fasten with Velcro® tab.
- 6. Note:** Slip one finger beneath each strap to ensure the fabric is not so tight that it impedes circulation or infusion of IV fluids.

**For Detailed Directions**  
[www.ivhouse.com/how](http://www.ivhouse.com/how)



Manufactured for I.V. House, Inc.  
2072 Concourse Dr.  
St. Louis, MO 63146 USA  
800-530-0400 (Toll Free)  
314-453-9200 (Phone)  
314-453-9576 (Fax)

Patents Pending  
All Rights Reserved  
Made in USA  
[www.ivhouse.com](http://www.ivhouse.com)  
[IVinfo@ivhouse.com](mailto:IVinfo@ivhouse.com)



Wellkang Ltd, Enterprise Hub,  
NW Business Complex, 1 Beraghmore Rd,  
Derry, BT48 8SE, Northern Ireland

