


# I.V. House 959XL-Ultra TLC® Elbow Splint Directions


Single Use Only—Fits either arm.

The TLC Elbow Splint can be applied to the arm before or after inserting IV and connecting to extension set or IV tubing.


The 959XL-Ultra Elbow Splint will fit most patients who require a small adult or standard adult size blood pressure cuff. Fits arms 24–34 cm (9.4–13.4 in) circumference at upper arm

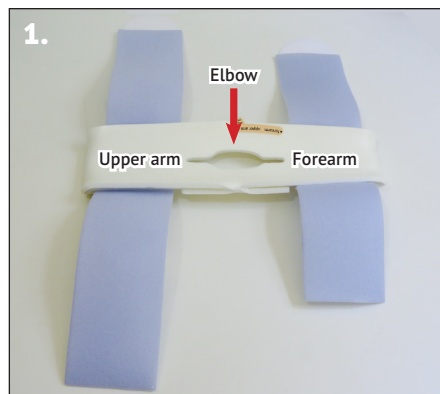


1. Remove splint from package and unwrap straps. With foam side up, note orientation indicated on label. Then peel label away and discard.

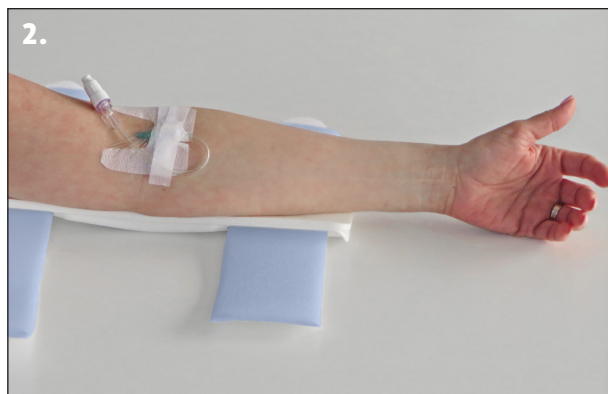


2. With patient's thumb pointing up, fit foam surface of splint to patient's arm. Align point of patient's elbow to center opening in splint. Place wide end of splint to patient's upper arm, narrow end at forearm.

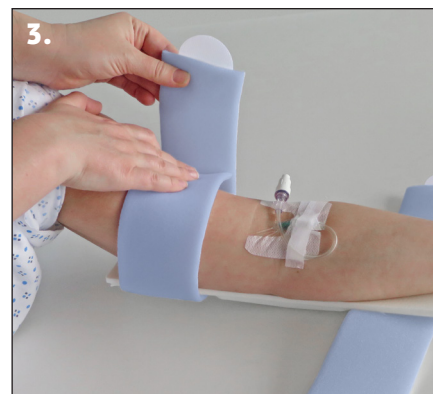


3. Wrap strap at wide end of splint around patient's upper arm.

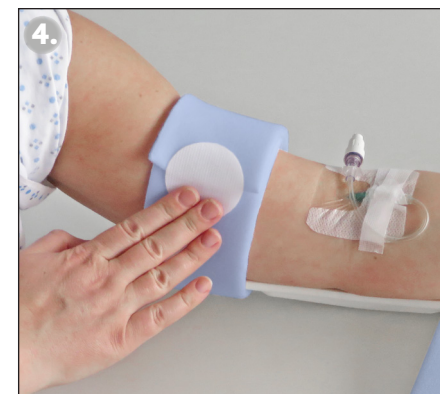

4. Secure strap with Velcro® coin.

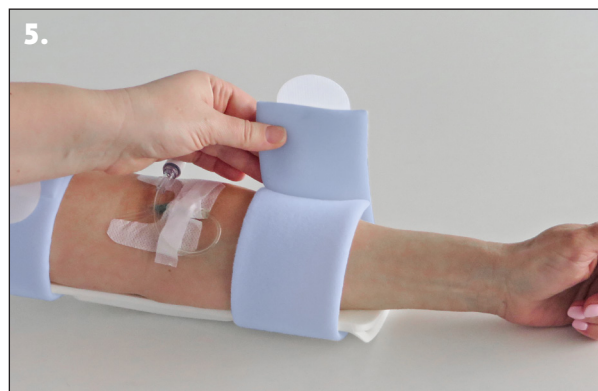

5. Wrap strap at narrow end of splint around patient's forearm.

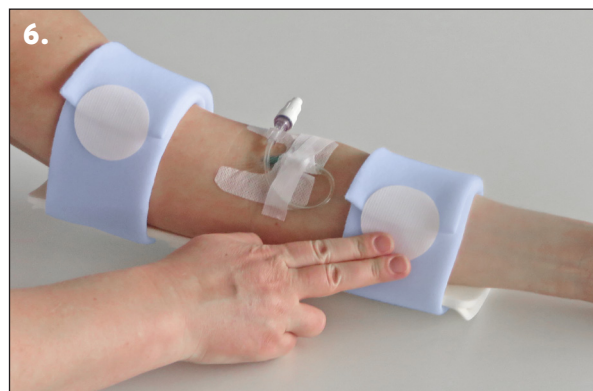

6. Secure strap with Velcro® coin. (Strap extenders are available separately for arms of greater diameter.)



7. Protect I.V. insertion site with a 730Arm I.V. House UltraDressing®.