

# FaceShield with ID Band

Reusable , Full-Length, Anti-Fog FaceShield  
with ID Band, and Strap with Velcro Closure

Fits most adults. Can be reused up to 50 times.  
See inside for detailed information on reuse.



(01)00686004000774

CATALOG ITEM:

**REF** 880FS-ID/50

**LOT** 2020-10-01-880FS-ID REV 1020



## Donning and Doffing

Wearable with glasses, goggles, masks, N95 respirator masks, and half facepiece elastomeric respirators.

Adjust fabric strap to fit your head, first use only. No need to undo strap at the Velcro® when removing FaceShield.

When doffing, grab rear strap and pull it up and over the head, gently allowing FaceShield to fall forward. Perform disinfection.\* Wash hands.

When donning after initial adjustment, simply pull strap over head.


\*See inside for detailed information on cleaning and disinfection, storage, and reuse parameters for the I.V. House FaceShield with ID Band.

ID Band for adding name or other information

Adjustable strap with Velcro® hook attachment

Foam cushion at forehead

Optically clear fog-resistant shield

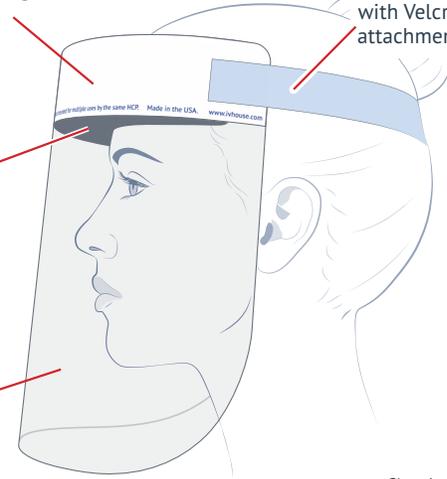

Cleaning and care  
[www.ivhouse.com/fs](http://www.ivhouse.com/fs)



MRI SAFE



LATEX FREE



DEHP  
DEHP-FREE



BPA FREE



FOR ONE  
WEARER

Manufactured for I.V. House, Inc.  
St. Louis, MO 63146 USA  
[IVinfo@ivhouse.com](mailto:IVinfo@ivhouse.com) (E-mail)

Made in USA

800-530-0400 (Toll Free)  
314-453-9200 (Phone)  
314-453-9576 (Fax)

[www.ivhouse.com](http://www.ivhouse.com)



**I.V. HOUSE®**  
Protection Over and Above®

## Cleaning and Disinfection

1. Prepare an area for disinfection of the FaceShield by first cleaning and sanitizing a table or counter surface.
2. Carefully wipe the inside and then the outside of the shield using a clean cloth saturated with neutral detergent solution. Rinse if needed.
3. Carefully wipe the outside of the shield using a clean cloth or wipe saturated with germicidal disinfectant solution; be sure shield remains wet for the required contact time. Avoid using the germicidal wipe on the foam and strap. DO NOT use germicidal disinfectant on the inside of the face shield as it can damage anti-fog coating agent.
4. Wipe the outside of the shield with clean water to remove residue.
5. Dry completely (air dry or use clean absorbent towels).
6. Rest the cleaned/disinfected FaceShield on the clean surface.
7. Doff gloves, perform hand hygiene.
8. Store disinfected FaceShield in paper bag provided (see below) in a locker for later reuse or as directed by your health care facility policy and procedures.
9. Perform hand hygiene after removing the FaceShield and before putting it on again if your workplace is practicing reuse.

We recommend following CDC guidelines or your facility's policies and procedures for donning and doffing reusable FaceShields and other personal protective equipment. For more information on CDC guidelines, please visit [www.cdc.gov/coronavirus/2019-ncov/hcp/using-ppe.html](http://www.cdc.gov/coronavirus/2019-ncov/hcp/using-ppe.html)

\*Facilities implementing reuse or extended use of PPE will need to adjust their donning and doffing procedures to accommodate those practices.

## Storage

Each FaceShield with ID Band comes individually packaged with a paper bag suitable for storing the device and other PPE in a work locker. We recommend recording each reuse with a tally mark system on the storage bag or ID band.

Store in a cool, dry location out of sunlight. To prevent damage, the FaceShield should be stored in its original shape and form. Do not roll or bend.

Write-on ID Band and provided paper storage bag make it easy to keep PPE separate by health care professional between uses.

## Reuse Guidelines and Considerations

FaceShields should be dedicated to a single, individual health care professional.

Disinfection of the FaceShield is required between uses. Fogging should not occur if cleaned as directed. If clarity of shield is reduced when worn, wipe off the outer side of the device with a soft, damp cloth to remove any disinfectant residue.

The following usage recommendations are based on findings in simulated use tests performed by Community Health Associates, LLC.

- If using a **bleach wipe** or similar solution for disinfection, the recommended repeated uses is **no greater than 30 uses**.
- If using a **hydrogen peroxide wipe** or similar solution for disinfection, the recommended repeated uses is **no greater than 50 uses**.

**Please note:** The reuse results listed above are estimates. Environmental, reprocessing, and individual user factors may impact number of uses per FaceShield.

---

## Testimonial

*"As a COVID-19 first responder, having immediate access to high-quality PPE is a top priority for me and my colleagues that are caring for these highly infectious patients. The CDC recommends the use of both respiratory and eye protection, and many commercially available solutions simply are not durable enough to withstand regular use by healthcare providers. This FaceShield with ID Band remained usable for 50 uses while taking care of COVID-19 Patients without issue and was extremely comfortable for me to wear during a rigorous 12-hour shift. I would highly recommend this product to others looking for ready to use PPE solutions."*

— Dr. J. Hudson Garrett, Jr.  
President and Chief Executive Officer  
Community Health Associates, LLC.



Manufactured for I.V. House, Inc.  
St. Louis, MO 63146 USA  
[Ivinfo@ivhouse.com](mailto:Ivinfo@ivhouse.com) (E-mail)

Made in USA

800-530-0400 (Toll Free)  
314-453-9200 (Phone)  
314-453-9576 (Fax)

[www.ivhouse.com](http://www.ivhouse.com)

