


6.

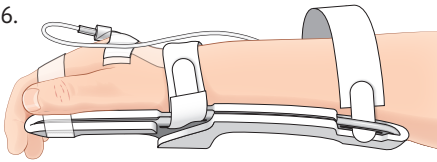


Wrap strap around forearm. Thread Velcro tab through slot at forearm. Pull the tab through the slot, fold the strap back around the outer edge of the splint onto itself, and secure it with the Velcro tab.

Note: Slip one finger beneath each strap to ensure the fabric is not so tight that it impedes circulation or infusion of IV fluids.

7.

Optional Strap Configuration

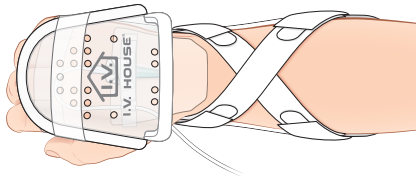

If the wrist strap obscures the catheter insertion site, consider a crisscross strap configuration instead.

Put the wrist strap through the forearm slot and the forearm strap through the wrist slot. This will allow easy visual inspection of the catheter insertion site.

Notes: Protect the catheter and loop of tubing with an I.V. House UltraDressing® or I.V. House UltraDome®.

Check IV site hourly for redness, swelling, and tenderness. **Touch, Look, and Compare** the IV site with the opposite palm and forearm. Change the TLC Wrist Splint each time a new IV is inserted.

TLC® Wrist Splint with Straps

Touch/Look/Compare



939M-Ultra

1 Medium (Toddler) Size Foam Padded Wrist Splint with Two Straps and Velcro® Closure

Fits most toddlers ages 12-36 months, or weighing 9-15 kg

CATALOG ITEM:

REF 939M-Ultra



(01)00686004000118

LOT 2019-07-01-939M-Ultra REV 0719



Video Directions: www.ivhouse.com/how

Manufactured for I.V. House, Inc.
 St. Louis, MO 63146 USA
IVinfo@ivhouse.com (E-mail)
www.ivhouse.com
 800-530-0400 (Toll Free)
 314-453-9200 (Phone)
 314-453-9576 (Fax)

Patents Pending
 All Rights Reserved
 Made in USA

EC REP Wellkang Ltd (www.ce-marking.eu)
 29 Harley St., London W1G 9QR, UK



I.V. HOUSE®
 Protection Over and Above®