


6.

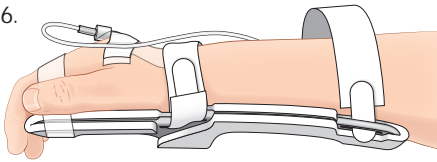


Wrap strap around forearm. Thread Velcro tab through slot at forearm. Pull the tab through the slot, fold the strap back around the outer edge of the splint onto itself, and secure it with the Velcro tab.

**Note:** Slip one finger beneath each strap to ensure the fabric is not so tight that it impedes circulation or infusion of IV fluids.

7.

Optional Strap Configuration

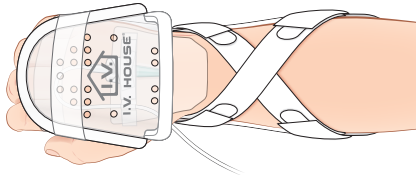

If the wrist strap obscures the catheter insertion site, consider a crisscross strap configuration instead.

Put the wrist strap through the forearm slot and the forearm strap through the wrist slot. This will allow easy visual inspection of the catheter insertion site.

**Notes:** Protect the catheter and loop of tubing with an I.V. House UltraDressing® or I.V. House UltraDome®.

**Check IV site hourly** for redness, swelling, and tenderness. **Touch, Look, and Compare** the IV site with the opposite palm and forearm. Change the TLC Wrist Splint each time a new IV is inserted.

## TLC® Wrist Splint with Straps

Touch/Look/Compare



### 939S-W-Ultra

1 Small (Infant) Size Foam Padded Wrist Splint with Two Straps and Velcro® Closure

Fits most infants ages 0–12 months, or weighing 2.5–12 kg

CATALOG ITEM:

**REF** 939S-W-Ultra



(01)00686004000460

**LOT** 2020-01-01-939S-W-Ultra REV 0120



Video Directions: [www.ivhouse.com/how](http://www.ivhouse.com/how)

Manufactured for I.V. House, Inc.  
 St. Louis, MO 63146 USA  
 IVinfo@ivhouse.com (E-mail)  
[www.ivhouse.com](http://www.ivhouse.com)  
 800-530-0400 (Toll Free)  
 314-453-9200 (Phone)  
 314-453-9576 (Fax)

Patents Pending  
 All Rights Reserved  
 Made in USA

**EC REP** Wellkang Ltd ([www.ce-marking.eu](http://www.ce-marking.eu))  
 29 Harley St., London W1G 9QR, UK



**I.V. HOUSE®**  
 Protection Over and Above®

