



**I.V. HOUSE®**

Protection Over and Above®



# 330M I.V. House UltraDressing®

**1 Smaller Dome Size IV Site Protector - 330 Series**

**1 Sheet of 4 Puppy Stickers**

U.S. Patents Nos.

6,526,981 / 8,006,699 / 8,640,707

Canadian Patent No. 2,415,102

Additional Patents Pending

All Rights Reserved • Made in USA

Video Directions and Info:

[www.ivhouse.com/how](http://www.ivhouse.com/how)

Manufactured for I.V. House, Inc.

2072 Concourse Dr.

St. Louis, MO 63146 USA

[IVinfo@ivhouse.com](mailto:IVinfo@ivhouse.com) (E-mail)

800-530-0400 (toll free)

314-453-9200 (phone)

314-453-9576 (fax)



Wellkang Ltd, Enterprise Hub,  
NW Business Complex, 1 Beraghmore Rd,  
Derry, BT48 8SE, Northern Ireland



SINGLE USE



MRI SAFE



LATEX FREE



DEHP  
DEHP FREE



BPA FREE

File REV 2023-05-02

CATALOG ITEM: **REF** 330M



REV



(01)0068600400071



**I.V. HOUSE®**

Protection Over and Above®

M

# I.V. House UltraDressing® Directions

## Single Use Only

1. Protect the loop of 3" extension tubing and catheter hub with the plastic dome.
2. Wrap comfortably leaving enough stretch to fit a finger under the fabric.
3. Cut off excess fabric.
4. Rewrap the fabric. Close with Velcro®.
5. Inspect the IV site by releasing the Velcro tab, lifting the dome, and viewing the entire site.
6. Change the I.V. House UltraDressing for each new IV insertion or when soiled.

## Note:

It is important that the looped tubing is covered by the dome to prevent accidental snagging of the loop and catheter hub. This also prevents bending and occlusion. Covering both the hub and the looped tubing virtually eliminates catheter manipulation.

## Puppy Sticker

Remove the crack and peel backing from the puppy sticker and apply the sticker to the dome. Tell the patient that the puppy likes to sleep on the "house" and to leave her alone.

