



I.V. HOUSE®

Protection Over and Above®



330M I.V. House UltraDressing®

1 Smaller Dome Size IV Site Protector - 330 Series

1 Sheet of 4 Puppy Stickers

Directions for use on back of insert

Manufactured for I.V. House, Inc., 2072 Concourse Dr., St. Louis, MO 63146 USA

IVinfo@ivhouse.com (E-mail)

www.ivhouse.com

800-530-0400 (Toll Free)

314-453-9200 (Phone)

314-453-9576 (Fax)

All Rights Reserved

Made in USA



(01)00686004000071



Wellkang Ltd, Enterprise Hub,
NW Business Complex, 1 Beraghmore Rd,
Derry, BT48 8SE, Northern Ireland



MEDICAL DEVICE

CATALOG ITEM: **REF** 330M

LOT 2025-07-01-330M REV 0725



SINGLE USE



MRI SAFE



LATEX FREE



DEHP
DEHP FREE



BPA FREE

330M I.V. House UltraDressing®

Directions for Use— Single Use Only

1. **Protect the loop of 3” extension tubing** and catheter hub with the plastic dome.
2. **Wrap comfortably**, leaving enough stretch to fit a finger under the fabric.
3. **Cut off excess fabric.**
4. **Rewrap the fabric.** Close with Velcro®.
5. **Inspect the IV site** by releasing the Velcro tab, lifting the dome, and viewing the entire site.
6. **Change the I.V. House UltraDressing** for each new IV insertion or when soiled.

Puppy Sticker

Remove the crack and peel backing from the puppy sticker and apply the sticker to the dome.

Tell the patient that the puppy likes to sleep on the “house” and to leave her alone.

Note:

It is important that the looped tubing is covered by the dome to prevent accidental snagging of the loop and catheter hub. This also prevents bending and occlusion. Covering both the hub and the looped tubing virtually eliminates catheter manipulation.



For Detailed Directions
www.ivhouse.com/how