



I.V. HOUSE®

Protection Over and Above®

727SFP

727SFP I.V. House UltraDome® IV Site Protector

1 Small Foam Padded Dome IV Site Protector • 1 Sheet of 4 Puppy Stickers

Directions for use on back of insert

Manufactured for I.V. House, Inc., St. Louis, MO 63146 USA

IVinfo@ivhouse.com (E-mail)

www.ivhouse.com

800-530-0400 (Toll Free)

314-453-9200 (Phone)

314-453-9576 (Fax)

All Rights Reserved

Made in USA



(01)00686004000019



Wellkang Ltd (www.ce-marking.eu)
29 Harley St., London W1G 9QR, UK



CATALOG ITEM: REF 727SFP

LOT 2018-04-01-727SFP REV 0418



SINGLE USE



MRI SAFE



LATEX FREE



DEHP
FREE



BPA FREE



I.V. HOUSE®

Protection Over and Above®

727SFP

727SFP I.V. House UltraDome® Directions

Single Use Only

1. Place an I.V. House UltraDome over the catheter hub and transparent dressing after taping the loop of tubing or extension set small enough to fit entirely under the inner curve of the domed end of the I.V. House UltraDome. Tape across the I.V. House UltraDome to hold the protective dome in place. Allow the tubing to exit the open end.
2. Lift the tape from one side to inspect the IV site according to your nursing practice guidelines.
3. Change the I.V. House UltraDome each time a new IV is inserted.

Puppy Sticker

Remove the crack and peel backing from the puppy sticker and apply the sticker to the dome. Tell the patient that the puppy likes to sleep on the “house” and to leave her alone.



Video directions and info:
www.ivhouse.com/how