



I.V. HOUSE®

Protection Over and Above®

727SFP

727SFP I.V. House UltraDome® IV Site Protector

1 Small Foam Padded Dome IV Site Protector • 1 Sheet of 4 Puppy Stickers

Directions for use on back of insert

Manufactured for I.V. House, Inc., 2072 Concourse Dr., St. Louis, MO 63146 USA

IVinfo@ivhouse.com (E-mail)

www.ivhouse.com

800-530-0400 (Toll Free)

314-453-9200 (Phone)

314-453-9576 (Fax)

All Rights Reserved

Made in USA



(01)00686004000019



Wellkang Ltd, Enterprise Hub,
NW Business Complex, 1 Beraghmore Rd,
Derry, BT48 8SE, Northern Ireland



MEDICAL DEVICE

CATALOG ITEM: **REF** 727SFP

LOT 2025-04-01-727SFP REV 0425



SINGLE USE



MRI SAFE



LATEX FREE



DEHP
DEHP FREE



BPA FREE

727SFP I.V. House UltraDome® Directions

Single Use Only

1. Following IV insertion, **tape the loop of tubing** small enough to fit entirely inside the inner curve of the dome.
2. **Place an I.V. House UltraDome** over the catheter hub and transparent dressing.
3. **Tape across the I.V. House UltraDome** to hold the protective dome in place. Allow the tubing to exit the open end.
4. Lift the tape from one side to **inspect the IV site** according to your nursing practice guidelines.
5. **Change the I.V. House UltraDome** each time a new IV is inserted.

Puppy Sticker

Remove the crack and peel backing from the puppy sticker and apply the sticker to the dome.

Tell the patient that the puppy likes to sleep on the “house” and to leave her alone.



For Detailed Directions
www.ivhouse.com/how