



**I.V. HOUSE®**

Protection Over and Above®

Arm

# 730Arm I.V. House UltraDressing®

**1 Arm Size IV Site Protector - 730 Series**  
**1 Puppy Sticker**

U.S. Patents Nos.  
6,526,981 / 8,006,699 / 8,640,707  
Canadian Patent No. 2,415,102  
Additional Patents Pending  
All Rights Reserved • Made in USA

Video Directions and Info:  
[www.ivhouse.com/how](http://www.ivhouse.com/how)

Manufactured for I.V. House, Inc.  
2072 Concourse Dr.  
St. Louis, MO 63146 USA  
[IVinfo@ivhouse.com](mailto:IVinfo@ivhouse.com) (E-mail)

800-530-0400 (toll free)  
314-453-9200 (phone)  
314-453-9576 (fax)



Wellkang Enterprise Hub,  
NW Business Complex, 1 Beraghmore Rd,  
Derry, BT48 8SE, Northern Ireland.



SINGLE USE



MRI SAFE



LATEX FREE



DEHP  
DEHP FREE



BPA FREE

CATALOG ITEM: **REF** 730Arm

**LOT** 2022-07-01-730Arm REV0722



(01)0068600400057



**I.V. HOUSE**

Protection Over and Above

Arm

## I.V. House UltraDressing® Directions

### Single Use Only - Fits Either Arm

1. Protect the loop of tubing and catheter hub with the plastic dome.
2. Wrap comfortably leaving enough stretch to fit a finger under the fabric.
3. Cut off excess fabric.
4. Remove the blue release paper to expose a strip of soft surgical tape that keeps the device from sliding. Rewrap the fabric gently pressing the tape into place. Close with Velcro®.
5. Inspect the IV site by releasing the Velcro tab, lifting the dome, and viewing the entire site.
6. Change the I.V. House UltraDressing each time a new IV is inserted.

### Note:

It is important that the looped tubing is covered by the dome to prevent accidental snagging of the loop and catheter hub. This also prevents bending and occlusion. Covering both the hub and the looped tubing virtually eliminates catheter manipulation.

### Puppy Sticker

Remove the crack and peel backing from the puppy sticker and apply the sticker to the dome. Tell the patient that the puppy likes to sleep on the "house" and to leave her alone.

