



I.V. HOUSE®

Protection Over and Above®

750LFP

750LFP I.V. House UltraDome® Plus IV Site Protector

**1 Large Foam Padded Dome IV Site Protector Plus Open Edge Padding
1 Puppy Sticker**

Directions for use on back of insert

Manufactured for I.V. House, Inc., 2072 Concourse Dr., St. Louis, MO 63146 USA

IVinfo@ivhouse.com (E-mail)

www.ivhouse.com

800-530-0400 (Toll Free)

314-453-9200 (Phone)

314-453-9576 (Fax)

All Rights Reserved

Made in USA



(01)00686004000323



Wellkang Enterprise Hub,
NW Business Complex, 1 Beraghmore Rd,
Derry, BT48 8SE, Northern Ireland.



CATALOG ITEM: **REF** 750LFP

LOT 2022-07-01-750LFP REV 0722



SINGLE USE



MRI SAFE



LATEX FREE



DEHP
FREE



BPA FREE



I.V. HOUSE®

Protection Over and Above®

750LFP

750LFP I.V. House UltraDome® Plus Directions

Single Use Only

1. Place an I.V. House UltraDome over the catheter hub and transparent dressing after taping the loop of tubing or extension set small enough to fit entirely under the inner curve of the domed end of the I.V. House UltraDome. Tape across the I.V. House UltraDome to hold the protective dome in place. Allow the tubing to exit the open end.
2. Lift the tape from one side to inspect the IV site according to your nursing practice guidelines.
3. Change the I.V. House UltraDome each time a new IV is inserted.

Puppy Sticker

Remove the crack and peel backing from the puppy sticker and apply the sticker to the dome. Tell the patient that the puppy likes to sleep on the “house” and to leave her alone.



Video directions and info:
www.ivhouse.com/how