



**Notes:** Protect the catheter and loop of tubing with an I.V. House UltraDressing® or I.V. House UltraDome®. Check IV site hourly for redness, swelling, and tenderness. **Touch, Look, and Compare** the IV site with the opposite palm and forearm. Change the TLC Wrist Splint each time a new IV is inserted.



Video Directions: [www.ivhouse.com/how](http://www.ivhouse.com/how)

Manufactured for I.V. House, Inc.  
 2072 Concourse Dr.  
 St. Louis, MO 63146 USA  
 IVinfo@ivhouse.com (E-mail)  
[www.ivhouse.com](http://www.ivhouse.com)  
 800-530-0400 (Toll Free)  
 314-453-9200 (Phone)  
 314-453-9576 (Fax)

Patents Pending  
 All Rights Reserved  
 Made in USA



**WRIST / CHILD / 12-41 KG**

**TLC® Wrist Splint  
 with Straps**  
 Touch/Look/Compare



**939L-Ultra**

1 Large (Child) Size Foam Padded Wrist Splint  
 with Three Straps and Velcro® Closure

Fits most children ages 3–10 years,  
 or weighing 12–41 kg

CATALOG ITEM:

REF 939L-Ultra



LOT 2022-07-01-939L-Ultra REV 0722



**I.V. HOUSE®**  
 Protection Over and Above®

