


6.

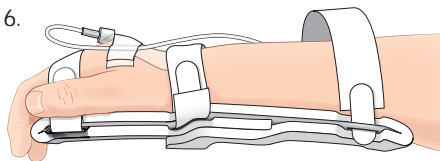


Wrap strap around forearm. Thread Velcro tab through slot at forearm. Pull the tab through the slot, fold the strap back around the outer edge of the splint onto itself, and secure it with the Velcro tab.

Note: Slip one finger beneath each strap to ensure the fabric is not so tight that it impedes circulation or infusion of IV fluids.

7.

Optional Strap Configuration

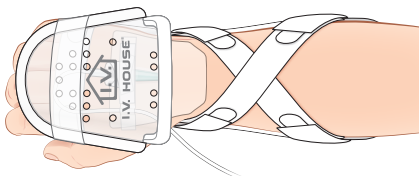

If the wrist strap obscures the catheter insertion site, consider a crisscross strap configuration instead.

Put the wrist strap through the forearm slot and the forearm strap through the wrist slot. This will allow easy visual inspection of the catheter insertion site.

Notes: Protect the catheter and loop of tubing with an I.V. House UltraDressing® or I.V. House UltraDome®.

Check IV site hourly for redness, swelling, and tenderness. **Touch, Look, and Compare** the IV site with the opposite palm and forearm. Change the TLC Wrist Splint each time a new IV is inserted.

WRIST / CHILD / 12-41 KG

TLC® Wrist Splint with Straps

Touch/Look/Compare



939L-W-Ultra

1 Large (Child) Size Foam Padded Wrist Splint with Three Straps and Velcro® Closure

Fits most children ages 3–10 years, or weighing 12–41 kg

CATALOG ITEM:

REF 939L-W-Ultra



(01)00686004000507

LOT 2020-01-01-939L-W-Ultra REV 0120



Video Directions: www.ivhouse.com/how

Manufactured for I.V. House, Inc.

St. Louis, MO 63146 USA

IVinfo@ivhouse.com (E-mail)

www.ivhouse.com

800-530-0400 (Toll Free)

314-453-9200 (Phone)

314-453-9576 (Fax)

Patents Pending

All Rights Reserved

Made in USA

EC REP Wellkang Ltd (www.ce-marking.eu)
29 Harley St., London W1G 9QR, UK



I.V. HOUSE®
Protection Over and Above®

