

WRIST / TODDLER / 9-15 KG

BASIC SPLINT

APPLY WITH TAPE

TLC® Wrist Splint

Touch/Look/Compare

Basic Splint–Apply with Tape

M

939M-Basic

1 Medium (Toddler) Size
Foam Padded Wrist Splint

Fits most toddlers ages 12–36 months,
or weighing 9–15 kg

CATALOG ITEM:

REF 939M-Basic



MEDICAL DEVICE



(01)00686004000125

LOT 2024-10-01-939M-Basic REV 1024



I.V. HOUSE®

Protection Over and Above®

TLC® Wrist Splint–Apply with Tape

Directions: Single Use Only—Fits either hand

1. **Apply 3 strips of anchor tape to the foam padding** on the back of the splint, one across the width of the splint at each end, and one across the center wrist support.
2. **The “fingers/wrist” label indicates proper placement.** Remove the label before applying the splint.
3. **Place the splint under the patient’s forearm**, with the heel of the hand landing in the slight bend of the device. The fingers should curl naturally over the end of the splint, leaving the hand in a functional position.
4. **Secure the splint with tape.** Place a ½” strip of tape across fingers and secure both ends to the anchor strip on the bottom of the splint. Fold under one end of tape to create a tab for easy removal.
5. **Repeat tape application at the wrist and forearm**, using double-backed tape to prevent epidermal stripping.
6. **Notes:** Do not cover openings on back side of the splint with tape. Covering the openings will obstruct the view of the palm and forearm.

For Detailed Directions
www.ivhouse.com/how



Manufactured for I.V. House, Inc.
2072 Concourse Dr.
St. Louis, MO 63146 USA
800-530-0400 (Toll Free)
314-453-9200 (Phone)
314-453-9576 (Fax)

U.S. Patent No. 12,213,904
Add'l Patents Pending
All Rights Reserved
Made in USA
www.ivhouse.com
IVinfo@ivhouse.com



SINGLE USE



MRI SAFE



LATEX FREE



DEHP
DEHP FREE



BPA FREE



Wellkang Ltd, Enterprise Hub,
NW Business Complex, 1 Beraghmore Rd,
Derry, BT48 8SE, Northern Ireland

