

**WRIST / TODDLER / 9-15 KG**

## TLC® Wrist Splint with Straps

Touch/Look/Compare



### 939M-W-Ultra

1 Medium (Toddler) Size Foam Padded Wrist Splint  
with Two Straps and Velcro® Closure

Fits most toddlers ages 12-36 months,  
or weighing 9-15 kg

CATALOG ITEM:

**REF** 939M-W-Ultra



MEDICAL DEVICE



(01)00686004000484

**LOT** 2025-04-01-939M-W-Ultra REV 0425



**I.V. HOUSE®**

Protection Over and Above®

# 939M-W-Ultra TLC® Wrist Splint

## Directions: Single Use Only—Fits either wrist

1. **Unroll the TLC Wrist Splint straps** and identify the smooth concave surface of the splint. This concave surface will be placed against the patient's hand and forearm.
2. **Turn the splint over and apply a strip of tape** to the foam on the back of the splint, near the end with no strap. This is an anchor tape strip to which all other tape will be applied.
3. **The "fingers/wrist" label indicates proper placement.** Remove the label before applying the splint.
4. **Place the splint under the patient's forearm**, with the heel of the hand landing in the slight bend in the device.
5. **Secure fingers by applying a half-inch strip of tape** to the anchor strip placed earlier. Wrap tape around fingers and back onto the anchor tape on the bottom. Fold under one end to create a tab for easy removal.
6. **Wrap strap around wrist.** Thread the Velcro tab into the wrist slot and pull it through. Fold the strap around the outer edge of the splint and back onto itself. Secure with Velcro tab.
7. **Repeat with forearm strap.** Secure with Velcro tab.
8. **Note:** Slip one finger beneath each strap to ensure the fabric is not so tight that it impedes circulation or infusion of IV fluids.
9. **If the wrist strap obscures the catheter insertion site**, consider a crisscross strap configuration instead. Put the wrist strap through the forearm slot and the forearm strap through the wrist slot.

**For Detailed Directions**

[www.ivhouse.com/how](http://www.ivhouse.com/how)



Manufactured for I.V. House, Inc.  
2072 Concourse Dr.  
St. Louis, MO 63146 USA  
800-530-0400 (Toll Free)  
314-453-9200 (Phone)  
314-453-9576 (Fax)

U.S. Patent No. 12,213,904  
Add'l Patents Pending  
All Rights Reserved  
Made in USA  
[www.ivhouse.com](http://www.ivhouse.com)  
[IVinfo@ivhouse.com](mailto:IVinfo@ivhouse.com)



SINGLE USE



MRI SAFE



LATEX FREE



DEHP  
DEHP FREE



BPA FREE



Wellkang Ltd, Enterprise Hub,  
NW Business Complex, 1 Beraghmore Rd,  
Derry, BT48 8SE, Northern Ireland

