

WRIST / YOUTH/ADULT / > 40 KG

TLC® Wrist Splint with Straps

Touch/Look/Compare



939XL-W-Ultra

1 Extra-Large (Youth/Adult) Size Foam Padded
Wrist Splint with Three Straps and Velcro® Closure

Fits most youth/adults 10 years and up,
or weighing over 40 kg

CATALOG ITEM:

REF 939XL-W-Ultra



MEDICAL DEVICE



(01)00686004000521

LOT 2025-07-01-939XL-W-Ultra REV 0725



I.V. HOUSE®

Protection Over and Above®

939XL-W-Ultra TLC® Wrist Splint

Directions: Single Use Only—Fits either wrist

1. **Unroll the TLC Wrist Splint straps** and identify the smooth concave surface of the splint. This concave surface will be placed against the patient's hand and forearm.
2. The **"fingers/wrist" label indicates proper placement**. Remove the label before applying the splint.
3. **Place the splint under the patient's forearm**, with the heel of the hand landing in the slight bend in the device.
4. **Wrap the first strap around the fingers**. Thread the Velcro® tab into slot and pull it through. Fold the strap around the outer edge of the splint and back onto itself. Secure with Velcro tab.
5. **Repeat with the other two straps**, securing each with the Velcro tab.
6. **Note:** Slip one finger beneath each strap to ensure the fabric is not so tight that it impedes circulation or infusion of IV fluids.
7. **If the wrist strap obscures the catheter insertion site**, consider a crisscross strap configuration instead. Put the wrist strap through the forearm slot and the forearm strap through the wrist slot.

For Detailed Directions
www.ivhouse.com/how



Manufactured for I.V. House, Inc.
2072 Concourse Dr.
St. Louis, MO 63146 USA
800-530-0400 (Toll Free)
314-453-9200 (Phone)
314-453-9576 (Fax)

U.S. Patent No. 12,213,904
Add'l Patents Pending
All Rights Reserved
Made in USA
www.ivhouse.com
IVinfo@ivhouse.com



SINGLE USE



MRI SAFE



LATEX FREE



DEHP
DEHP FREE



BPA FREE



Wellkang Ltd, Enterprise Hub,
NW Business Complex, 1 Beraghmore Rd,
Derry, BT48 8SE, Northern Ireland

