

FOOT / INFANT / 6-12 KG

TLC® Foot Splint

Touch/Look/Compare



949S-Foot

1 Small (Infant) Size Foam Padded Foot Splint
with Two Straps and Velcro® Closure

Fits most infants ages 4-12 months,
or weighing 6-12 kg

May fit larger infants, non-ambulatory only

CATALOG ITEM:

REF 949S-Foot



MEDICAL DEVICE



(01)00686004000262

LOT 2024-10-01-949S-Foot REV 1024



I.V. HOUSE®
Protection Over and Above®

949S TLC® Foot Splint—Directions

Single Use Only—Fits either foot, laterally or medially

1. **Orient the concave side of the splint facing upward** and unroll the straps.
2. **Align the infant's ankle with the splint's middle opening.** The heel should roughly coincide with the prominent curve at the outer edge of the splint.
3. **Rotate the splint to position it** laterally or medially, on a left or right foot and determine which end will be fit to the toe and which end will be fit to the lower leg.
4. **Note that each of the two strap anchor slots has two positions.** With the splint oriented as it will fit to the infant's foot, slide the strap to the top position in each slot.
5. **Leg Strap:** Slide strap under IV tubing and wrap the top strap around the lower leg and the splint. Secure tubing at lower leg with final pass of strap. Fasten with Velcro® tab.
6. **Toe Strap:** Wrap the bottom strap over the toes, under the loop of tubing, and slip the free end through the hole in the splint near the toes. Fasten with Velcro® tab.
7. **Note:** Slip one finger beneath each strap to ensure the fabric is not so tight that it impedes circulation or infusion of IV fluids.

For Detailed Directions
www.ivhouse.com/how



Manufactured for I.V. House, Inc.
2072 Concourse Dr.
St. Louis, MO 63146 USA
800-530-0400 (Toll Free)
314-453-9200 (Phone)
314-453-9576 (Fax)

U.S. Patent No. 12,213,904
Add'l Patents Pending
All Rights Reserved
Made in USA
www.ivhouse.com
IVinfo@ivhouse.com



Wellkang Ltd, Enterprise Hub,
NW Business Complex, 1 Beraghmore Rd,
Derry, BT48 8SE, Northern Ireland

