



I.V. HOUSE®

Protection Over and Above®

M

959M-Basic TLC® Elbow Splint

1 Medium (Toddler/Child) Size Foam Padded Elbow Splint

Fits most toddlers and children 12 months–10 years, or 9–41 kg

CATALOG ITEM: **REF** 959M-Basic

LOT 2017-10-01-959M-Basic REV 1017



(01)00686004000385

Directions for use on back of insert

Manufactured for I.V. House, Inc., St. Louis, MO 63146 USA

IVinfo@ivhouse.com (E-mail) • www.ivhouse.com

800-530-0400 (Toll Free)

314-453-9200 (Phone)

314-453-9576 (Fax)

Patents Pending

Made in USA



Wellkang Ltd (www.ce-marking.eu)

29 Harley St., London W1G 9QR, UK



SINGLE USE



MRI SAFE



LATEX FREE



DEHP
DEHP FREE



BPA FREE

I.V. House TLC® Elbow Splint

Touch/Look/Compare



Directions: Single Use Only – Fits either arm

The TLC Elbow Splint can be applied to the arm before or after inserting IV and connecting to extension set or IV tubing. This is a nursing decision based on your practice.

1. Place the splint under the patient's arm, with the narrow end of the splint toward the hand and the wide end above the elbow. The point of the elbow should align with the center opening.
2. Secure the splint with tape. Place a ½" strip of tape across upper arm and secure both ends to the bottom of the splint. Fold under one end of tape to create a tab for easy removal.
3. Repeat tape application at the forearm, using double-backed tape to prevent epidermal stripping.
4. Protect the catheter and loop of tubing with an I.V. House UltraDressing® or I.V. House UltraDome®.
5. Check the IV site hourly for redness, swelling, and tenderness. **Touch, Look, and Compare** the IV site with opposite extremity, including forearm and surrounding elbow area. Change the TLC Splint each time a new IV is inserted.

Notes:

Do not cover openings on back side of the splint with tape. Covering the openings will obstruct the view of the elbow and arm.

Video Directions

are available at ivhouse.com/how