

**ELBOW / TODDLER/CHILD / 9-41 KG**

**BASIC SPLINT**

**APPLY WITH TAPE**

## TLC<sup>®</sup> Elbow Splint

Touch/Look/Compare

Basic Splint—Apply with Tape



## 959M-Basic

1 Medium (Toddler/Child) Size  
Foam Padded Elbow Splint

Fits most toddlers and children ages  
12 months–10 years, or weighing 9–41 kg

CATALOG ITEM:

**REF** 959M-Basic



MEDICAL DEVICE



(01)00686004000385

**LOT** 2025-00-00-959M-Basic REV 0000



**I.V. HOUSE<sup>®</sup>**

**Protection Over and Above<sup>®</sup>**

# TLC® Elbow Splint–Apply with Tape

## Directions: Single Use Only—Fits either arm

1. **Apply 4 strips of anchor tape to the foam padding on the back of the splint:** one across the width of the splint at the narrow end, one down each side of the splint parallel to the center opening, and one across the width at the wide end of the splint.
2. **The “forearm/upper arm” label indicates proper placement.** Remove the label before applying the splint.
3. **Place the splint under the patient’s arm,** oriented as indicated by the removable label. The point of the elbow should align with the center opening.
4. **Secure the splint with tape.** Using double-backed tape to prevent epidermal stripping, place a ½” strip of tape across upper arm and secure both ends to the bottom of the splint. Fold under one end of tape to create a tab for easy removal.
5. **Repeat tape application at the forearm,** using double-backed tape.
6. **Notes:** Do not cover opening on back side of the splint with tape. Covering the opening will prevent manual and visual assessment of the extremity.

For Detailed Directions  
[www.ivhouse.com/how](http://www.ivhouse.com/how)



Manufactured for I.V. House, Inc.  
2072 Concourse Dr.  
St. Louis, MO 63146 USA  
800-530-0400 (Toll Free)  
314-453-9200 (Phone)  
314-453-9576 (Fax)

U.S. Patent No. 12,213,904  
Add'l Patents Pending  
All Rights Reserved  
Made in USA  
[www.ivhouse.com](http://www.ivhouse.com)  
[IVinfo@ivhouse.com](mailto:IVinfo@ivhouse.com)



SINGLE USE



MRI SAFE



LATEX FREE



DEHP  
DEHP FREE



BPA FREE



Wellkang Ltd, Enterprise Hub,  
NW Business Complex, 1 Beraghmore Rd,  
Derry, BT48 8SE, Northern Ireland

