

ELBOW / NEWBORN/INFANT / 2.5-12 KG

BASIC SPLINT

APPLY WITH TAPE

TLC[®] Elbow Splint

Touch/Look/Compare

Basic Splint—Apply with Tape



959S-Basic

1 Small (Newborn/Infant) Size

Foam Padded Elbow Splint

Fits most infants ages 0-12 months,
or weighing 2.5-12 kg

CATALOG ITEM:

REF 959S-Basic



(01)00686004000378

LOT 2020-00-00-959S-Basic REV 0000



I.V. HOUSE[®]

Protection Over and Above[®]

TLC® Elbow Splint–Apply with Tape

Directions: Single Use Only - Fits either arm

The TLC Elbow Splint can be applied to the arm before or after inserting IV and connecting to extension set or IV tubing. This is a nursing decision based on your practice.

1. Place the splint under the patient's arm, with the narrow end of the splint toward the hand and the wide end above the elbow. The point of the elbow should align with the center opening.
2. Secure the splint with tape. Place a ½" strip of tape across upper arm and secure both ends to the bottom of the splint. Fold under one end of tape to create a tab for easy removal.
3. Repeat tape application at the forearm, using double-backed tape to prevent epidermal stripping.
4. Protect the catheter and loop of tubing with an I.V. House UltraDressing® or I.V. House UltraDome®.
5. Check the IV site hourly for redness, swelling, and tenderness. **Touch, Look, and Compare** the IV site with opposite extremity, including forearm and surrounding elbow area. Change the TLC Splint each time a new IV is inserted

Notes: Do not cover openings on back side of the splint with tape. Covering the openings will obstruct the view of the elbow and arm.

Video Directions are available at ivhouse.com/how

Manufactured for I.V. House, Inc.

2072 Concourse Dr.
St. Louis, MO 63146 USA
800-530-0400 (Toll Free)
314-453-9200 (Phone)
314-453-9576 (Fax)

Patents Pending
All Rights Reserved
Made in USA
www.ivhouse.com
IVinfo@ivhouse.com



Wellkang Enterprise Hub,
NW Business Complex, 1 Beraghmore Rd,
Derry, BT48 8SE, Northern Ireland.

