ELBOW / INFANT / 6-12 кс

TLC[®] Elbow Splint with Straps Touch/Look/Compare



1 Small (Infant) Size Foam Padded Elbow Splint with Three Straps and Velcro[®] Closure

Fits most infants ages 4–12 months, or weighing 6–12 kg Small splint, longer straps

CATALOG ITEM: REF 959S-Ultra



(01)00686004000347

LOT 2025-07-01-959S-Ultra REV 0725



TLC[®] Elbow Splint with Straps

Directions: Single Use Only-Fits either arm

- Unroll the TLC Elbow Splint straps and identify the smooth concave surface of the splint. This concave surface will be placed against the patient's arm.
- The "forearm/upper arm" label indicates proper placement. Remove the label before applying the splint.
- Place the splint under the patient's arm, with the narrow end of the splint toward the hand and the wide end above the elbow. The point of the elbow should align with the center opening. We recommend a thumbs-up application whenever possible.
- 4. Wrap the first strap around the forearm. Thread the Velcro[®] tab into slot and pull it through. Fold the strap around the outer edge of the splint and back onto itself. Secure with Velcro tab.
- 5. Repeat with the other two straps, securing each with the Velcro tab.
- Note: Slip one finger beneath each strap to ensure the fabric is not so tight that it impedes circulation or infusion of IV fluids.
- If the elbow strap obscures the catheter insertion site, consider a crisscross strap configuration instead. Put the elbow strap through the upper arm slot and the upper arm strap through the elbow slot.

For Detailed Directions www.ivhouse.com/how



Manufactured for I.V. House, Inc. 2072 Concourse Dr. St. Louis, MO 63146 USA 800-530-0400 (Toll Free) 314-453-9200 (Phone) 314-453-9576 (Fax)

U.S. Patent No. 12,213,904 Addt'l Patents Pending All Rights Reserved Made in USA www.ivhouse.com IVinfo@ivhouse.com





Wellkang Ltd, Enterprise Hub, NW Business Complex, 1 Beraghmore Rd, Derry, BT48 8SE, Northern Ireland CE