

**ELBOW / INFANT / 6-12 KG**

# TLC® Elbow Splint with Straps

Touch/Look/Compare



## 959S-W-Ultra

1 Small (Infant) Size Foam Padded Elbow Splint  
with Three Straps and Velcro® Closure

Fits most infants ages 4-12 months,  
or weighing 6-12 kg  
Small splint, longer straps

CATALOG ITEM:

**REF** 959S-W-Ultra



MEDICAL DEVICE



(01)00686004000545

**LOT** 2025-04-01-959S-W-Ultra REV 0425



**I.V. HOUSE®**

Protection Over and Above®

# TLC® Elbow Splint with Straps

## Directions: Single Use Only—Fits either arm

1. **Unroll the TLC Elbow Splint straps** and identify the smooth concave surface of the splint. This concave surface will be placed against the patient's arm.
2. The **"forearm/upper arm" label indicates proper placement**. Remove the label before applying the splint.
3. **Place the splint under the patient's arm**, with the narrow end of the splint toward the hand and the wide end above the elbow. The point of the elbow should align with the center opening. **We recommend a thumbs-up** application whenever possible.
4. **Wrap the first strap around the forearm**. Thread the Velcro® tab into slot and pull it through. Fold the strap around the outer edge of the splint and back onto itself. Secure with Velcro tab.
5. **Repeat with the other two straps**, securing each with the Velcro tab.
6. **Note:** Slip one finger beneath each strap to ensure the fabric is not so tight that it impedes circulation or infusion of IV fluids.
7. **If the elbow strap obscures the catheter insertion site**, consider a crisscross strap configuration instead. Put the elbow strap through the upper arm slot and the upper arm strap through the elbow slot.

For Detailed Directions  
[www.ivhouse.com/how](http://www.ivhouse.com/how)



Manufactured for I.V. House, Inc.  
2072 Concourse Dr.  
St. Louis, MO 63146 USA  
800-530-0400 (Toll Free)  
314-453-9200 (Phone)  
314-453-9576 (Fax)

U.S. Patent No. 12,213,904  
Add'l Patents Pending  
All Rights Reserved  
Made in USA  
[www.ivhouse.com](http://www.ivhouse.com)  
[IVinfo@ivhouse.com](mailto:IVinfo@ivhouse.com)



SINGLE USE



MRI SAFE



LATEX FREE



DEHP  
DEHP FREE



BPA FREE



Wellkang Ltd, Enterprise Hub,  
NW Business Complex, 1 Beraghmore Rd,  
Derry, BT48 8SE, Northern Ireland

