

ELBOW / NEWBORN / 2.5–7 kg

TLC® Elbow Splint with Straps

Touch/Look/Compare



959XS-Ultra

1 Extra-Small (Newborn) Size Foam Padded Elbow
Splint with Three Straps and Velcro® Closure

Fits most newborns ages 0–4 months,
or weighing 2.5–7 kg
Small splint, shorter straps

CATALOG ITEM:

REF 959XS-Ultra



(01)00686004000330

LOT 2024-10-01-959XS-Ultra REV 1024



I.V. HOUSE®
Protection Over and Above®

TLC® Elbow Splint with Straps

Directions: Single Use Only—Fits either arm

1. **Unroll the TLC Elbow Splint straps** and identify the smooth concave surface of the splint. This concave surface will be placed against the patient's arm.
2. The **"forearm/upper arm" label indicates proper placement.** Remove the label before applying the splint.
3. **Place the splint under the patient's arm**, with the narrow end of the splint toward the hand and the wide end above the elbow. The point of the elbow should align with the center opening. **We recommend a thumbs-up** application whenever possible.
4. **Wrap the first strap around the forearm.** Thread the Velcro® tab into slot and pull it through. Fold the strap around the outer edge of the splint and back onto itself. Secure with Velcro tab.
5. **Repeat with the other two straps**, securing each with the Velcro tab.
6. **Note:** Slip one finger beneath each strap to ensure the fabric is not so tight that it impedes circulation or infusion of IV fluids.
7. **If the elbow strap obscures the catheter insertion site**, consider a crisscross strap configuration instead. Put the elbow strap through the upper arm slot and the upper arm strap through the elbow slot.

For Detailed Directions
www.ivhouse.com/how



Manufactured for I.V. House, Inc.
2072 Concourse Dr.
St. Louis, MO 63146 USA
800-530-0400 (Toll Free)
314-453-9200 (Phone)
314-453-9576 (Fax)

U.S. Patent No. 12,213,904
Add'l Patents Pending
All Rights Reserved
Made in USA
www.ivhouse.com
IVinfo@ivhouse.com



SINGLE USE



MRI SAFE



LATEX FREE



DEHP
DEHP FREE



BPA FREE



Wellkang Ltd, Enterprise Hub,
NW Business Complex, 1 Beraghmore Rd,
Derry, BT48 8SE, Northern Ireland

